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- Radwanska, E., Frankenberg, J., and Allen, E., Plasma progesterone levels in normal and abnormal early human pregnancy. *Fertility and Sterility*, 1978; 30, 398-402.
- March, C.M., Goebelsmann, U., Nakamura, R.M., and Mishell, D.R. Jr., Roles of estradiol and progesterone in eliciting the midcycle luteinizing hormone and follicle-stimulating hormone surges, *J. Clin. Endo. Metab.*, 1979; 49, 507-513.
- Shepard, M.K., and Senturia, Y.D., Comparison of serum progesterone and endometrial biopsy for confirmation of ovulation and evaluation of luteal function. *Fertility and Sterility*, 1977; 28: 541-548.
- Ross, G.T., Vande Wiele, R.L., and Frantz, A.G., *The Ovaries and the breasts*. In: Williams, R.H., ed., *Textbook of Endocrinology*. Saunders Company, Philadelphia; 1981: 355-411.
- Tietz, N.W. ed., *Clinical Guide to Laboratory Tests*, 3rd Edition, W.B. Saunders, Co., Philadelphia, 1995: 509-512
- Johansson, E.D.B., and Jonasson, L.-E., Progesterone levels in amniotic fluid and plasma from women: I. Levels during normal pregnancy. *Acta Obstet. Gynec. Scand.*, 1971; 50: 339-343.
- USA Center for Disease Control/National Institute of Health Manual, "Biosafety in Microbiological and Biomedical Laboratories" 1984.

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